

PHYSITEMP INSTRUMENTS INC

154 Huron Avenue

Clifton NJ 07013

Tel: 1-800-452-8510

Email: info@[physitemp.com](mailto:info@physitemp.com)

Instructions for Sterilization of MT-series Flexible Microprobes

Cleaning before sterilization:

Wipe with alcohol or cleaning solution

Note: The stainless steel needle can be washed in an ultrasonic cleaner with enzymatic cleaner, but the plug should never be immersed in liquid. It is sealed with epoxy to prevent moisture entering during normal use and wiping, but has not been tested for immersion.

AUTOCLAVING: Sterilize at: 134°C for 3 minutes at 2.2 bar
or 121°C for 15 minutes
or 115°C for 30 minutes